

REAGENT	UNOPENED	OPENED	NOTES
<b>OraSure Oraquick Controls</b> (HIV 1–pos, HIV 2–pos, neg)	Listed expiration date when stored at: (2-8 °C) (35°-46 °F)	Expires in 8 weeks when stored at: (2-8°C) (35°-46°F) When you open a new set, write the new expiration date on the back of the box.	DO NOT STORE AT ROOM TEMPERATURE  DO NOT FREEZE
<b>OraSure Oraquick Advance</b> HIV 1 and 2 antibody	Listed expiration date when stored at: (2-27°C )(35°-80°F)	If individual kit package is opened and not used, discard immediately.	DO NOT STORE <35° or >80° F RUN QC IF STORAGE WAS OUT OF RANGE
REAGENT	UNOPENED	OPENED	NOTES
<b>Uni-Gold Recombigen Controls</b> (HIV 1–pos, HIV 2–pos, neg)	Listed expiration date when stored at: (2°-8°C) (36o-46°F)	Expires in 4 weeks when stored at: (2°-8°C) (36°-46°F)  When you open a new set, write the expiration date on the back of the box.	DO NOT STORE AT ROOM TEMPERATURE  DO NOT FREEZE
<b>Uni-Gold Recombigen</b> HIV 1 antibody	Listed expiration date when stored at: (2°-27°C )(36-81°F)	If individual kit package is opened and not used, discard immediately.	DO NOT STORE <36° or >80° F RUN QC IF STORAGE WAS OUT OF RANGE
REAGENT	UNOPENED	OPENED	NOTES
<b>Clearview Stat-Pak Controls</b> (HIV 1–pos, HIV 2–pos, neg)	Listed expiration date when stored at: (2°-8°C) (36°-46°F)	Manufacturer expiration date when stored at: (2°-8°C) (36°-46°F)  When you open a new set, write the expiration date on the back of the box.	DO NOT STORE AT ROOM TEMPERATURE  DO NOT FREEZE
<b>Clearview Stat-Pak</b> HIV 1 and 2 antibody	Listed expiration date when stored at: (8°-30°C )(46°-86°F)	If individual kit package is opened and not used, discard immediately.	DO NOT STORE <47 ° or >86° F  RUN QC IF STORAGE WAS OUT OF RANGE

<b>REAGENT</b>	<b>UNOPENED</b>	<b>OPENED</b>	<b>NOTES</b>
<b>Determine Controls</b>	Listed expiration date when stored at: (2°-8°C) (36°-46°F)	Manufacturer expiration date when stored at: (2°-8°C) (36°-46°F)  When you open a new set, write the expiration date on the back of the box.	DO NOT STORE AT ROOM TEMPERATURE  DO NOT FREEZE
<b>Determine HIV 1 and 2 Antigen/Antibody Combo</b>	Listed expiration date when stored at: (2°-30°C )(36°-86°F)	If stored refrigerated test units must be brought to operating temperatures (15°-30°C )(59°-86°F) Must be used within 2 hours after removing protective foil	DO NOT STORE <36 ° or >86° F  RUN QC IF STORAGE WAS OUT OF RANGE